Comp-Position

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Users guide

Your Comp-Position positioning chair is built for years of trouble-free service. Carefully read and follow the instructions step-by-step to assemble, operate and care for your chair.

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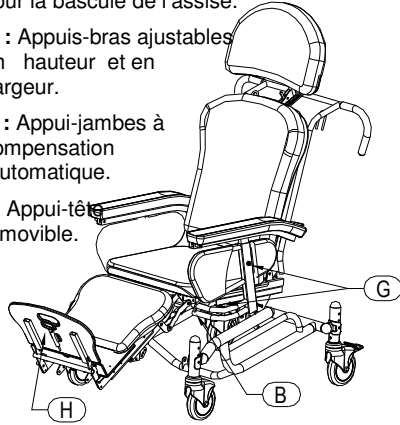
I : Appui-tête amovible.

Using the Comp-Position Chair

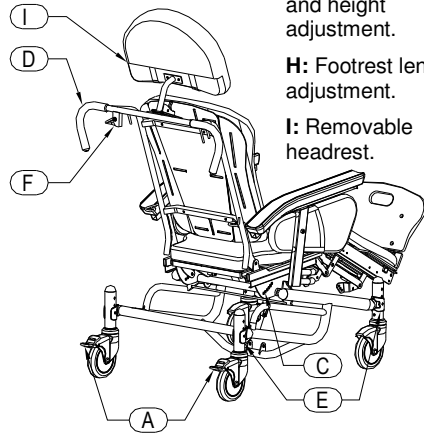
For the safest operation, please familiarize yourself with the following components:

- A**: Swivel wheels with brakes.
- B**: Seat depth adjustment.
- C**: Backrest angle adjustment.
- D**: Push bar.
- E**: Seat height adjustment.
- F**: Tilt-In-Space lever activator.

G: Armrest width and height adjustment.

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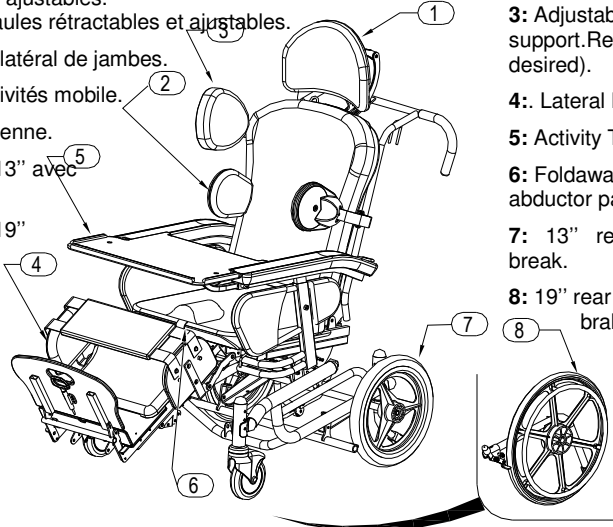
- 1: Headrest adjustment.
- 2: Structural Lateral Supports adjusts to provide comfortable, correct trunk positioning. Removable (if desired).
- 3: Adjustable shoulders support. Removable (if desired).
- 4: Lateral legs protector.
- 5: Activity Tray.
- 6: Foldaway perineal abductor pad.
- 7: 13" rear wheels with break.
- 8: 19" rear wheels with brake.

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THANK YOU FOR CHOOSING LPA MEDICAL PRODUCTS.

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PLEASE FILL OUR DETACH AND RETURN THIS INFORMATION CARD WITHIN 30 DAYS OF PURCHASE.

Date d'achat: _____ Date or purchase _____

Vendeur _____ Purchase from _____

Adresse _____ Address _____

Nom du centre _____ Facility's name _____

Adresse du centre _____ Facility's street address _____

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Nombre de lits _____ Number of beds in your facility _____

Sorte d'établissement _____ Facility type:

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What influence your decision the most

- Magazine Ad
- Direct mail
- Brand name
- Convention exhibit
- Friend's suggestion
- Salesperson's suggestion

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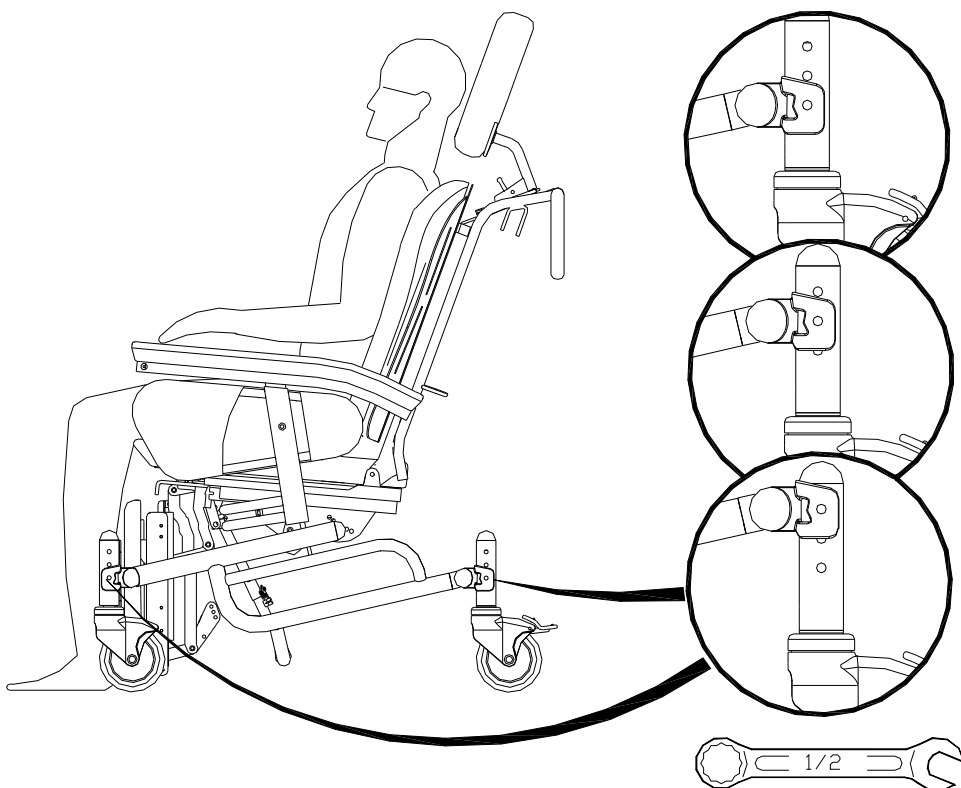
How to use of your *Comp-Position Chair*

All brakes **MUST BE ENGAGED AT ALL TIMES**, except during transportation.

1. Seat height adjustment.

- 1.1. When the chair is unoccupied, determine if the seat height should be adjusted up or down.
- 1.2. Unscrew and remove the 4 bolts on the seat height adjustment wheel structure as shown below.
- 1.3. Move the seat height adjustment wheel structure to the desired position. BE SURE EACH SIDE IS AT THE SAME LEVEL, tighten the 4 bolts securely with the lock washers in place.
- 1.4. Place the chair in an upright position and check if the seat is whell adapted for the occupant, correct if required.

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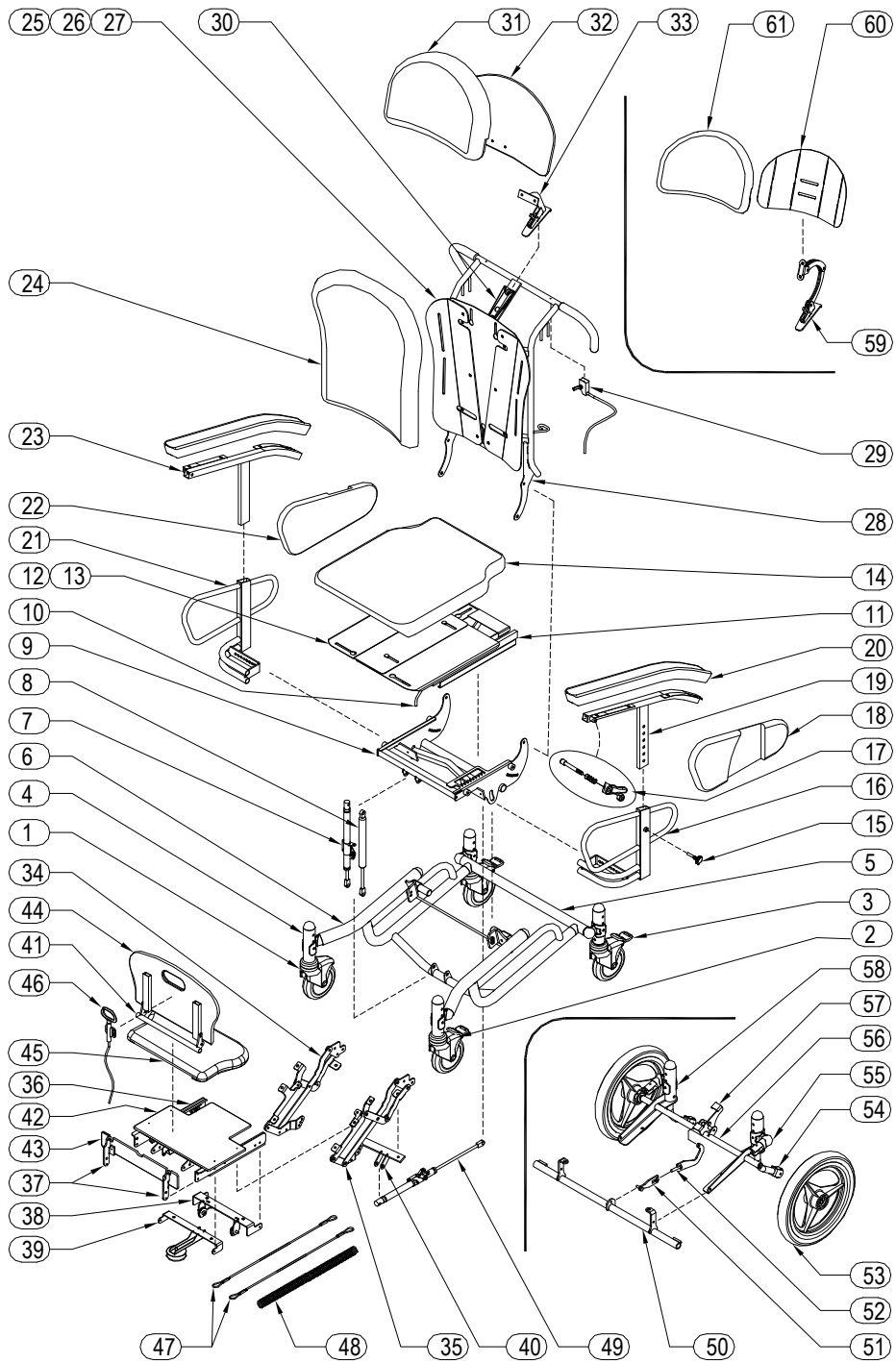
Always specify fabric name, pattern, color, and the Chair's serial number for all replacement part orders. Upholstered replacement parts may vary slightly in color from the original part due to dye lot differences. LPA Medical will not be responsible for dye lot differences

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
2. Seat depth.

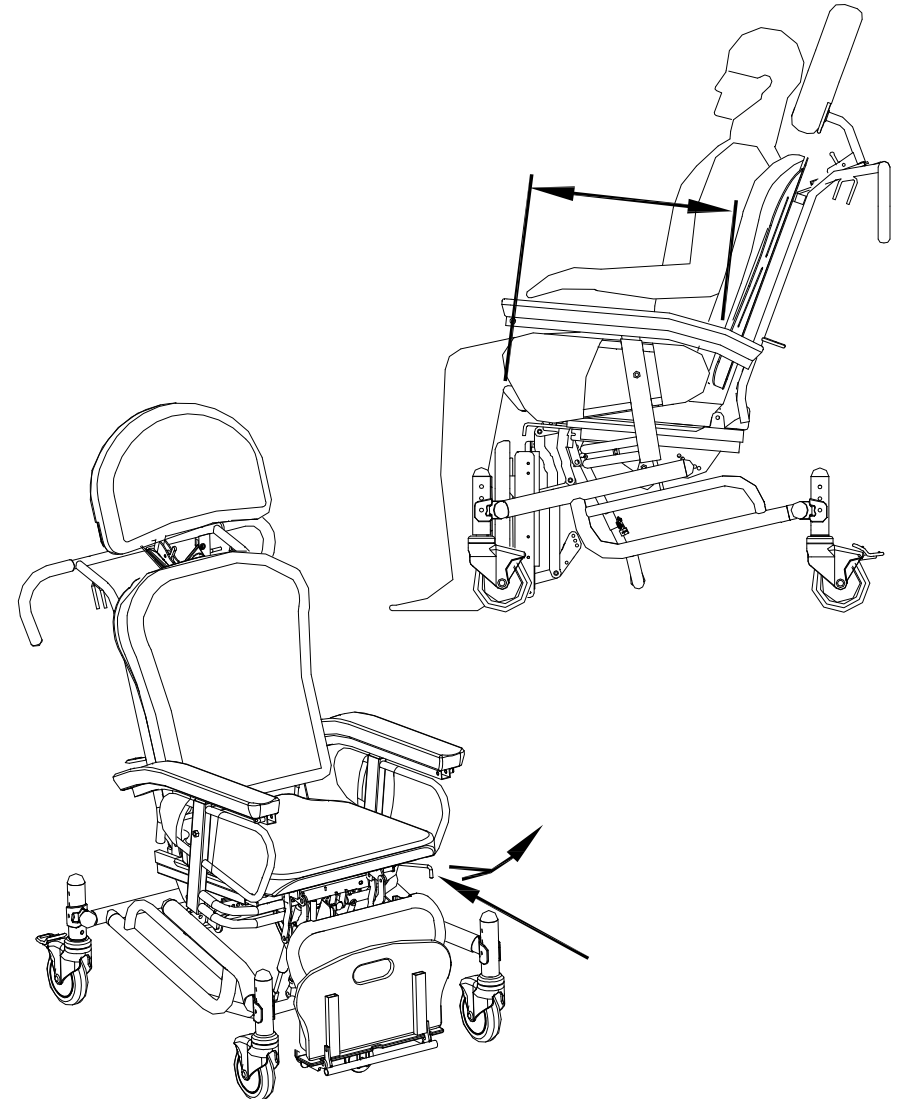- 2.1. Determine if the seat depth should be adjusted.
- 2.2. To adjust the seat depth, grip the handle behind the seat at front left, push it towards the left side and slide the seat to the correct position.
- 2.3. Check if the seat is well adapted for the occupant, correct if required.

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3. Seat/back angle adjustment.

- 3.1. When the chair is unoccupied, determine if the seat/back angle is positioned correctly.
 - 3.2. Remove the 2 bolts (1 per side) behind the chair, choose the correct angle.
 - 3.3. Tighten 2 bolts.
- BE SURE EACH SIDE IS AT THE SAME LEVEL.
- 3.4. Check if the seat is well adapted for the occupant, correct if required.

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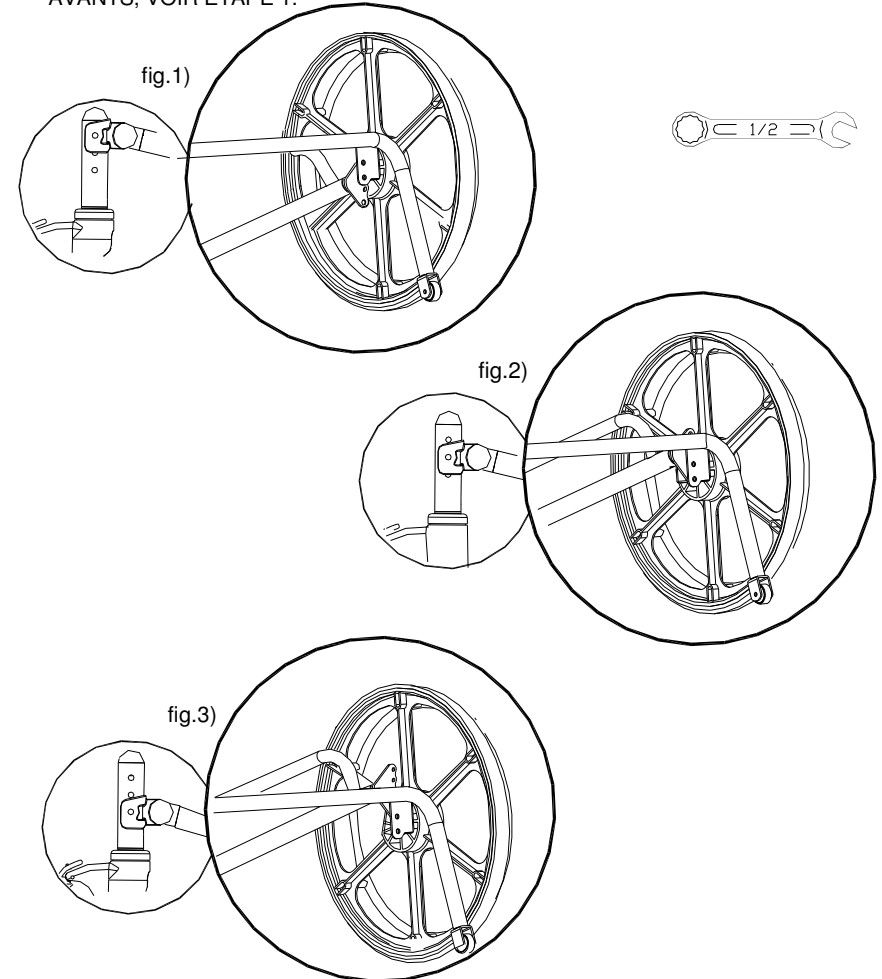
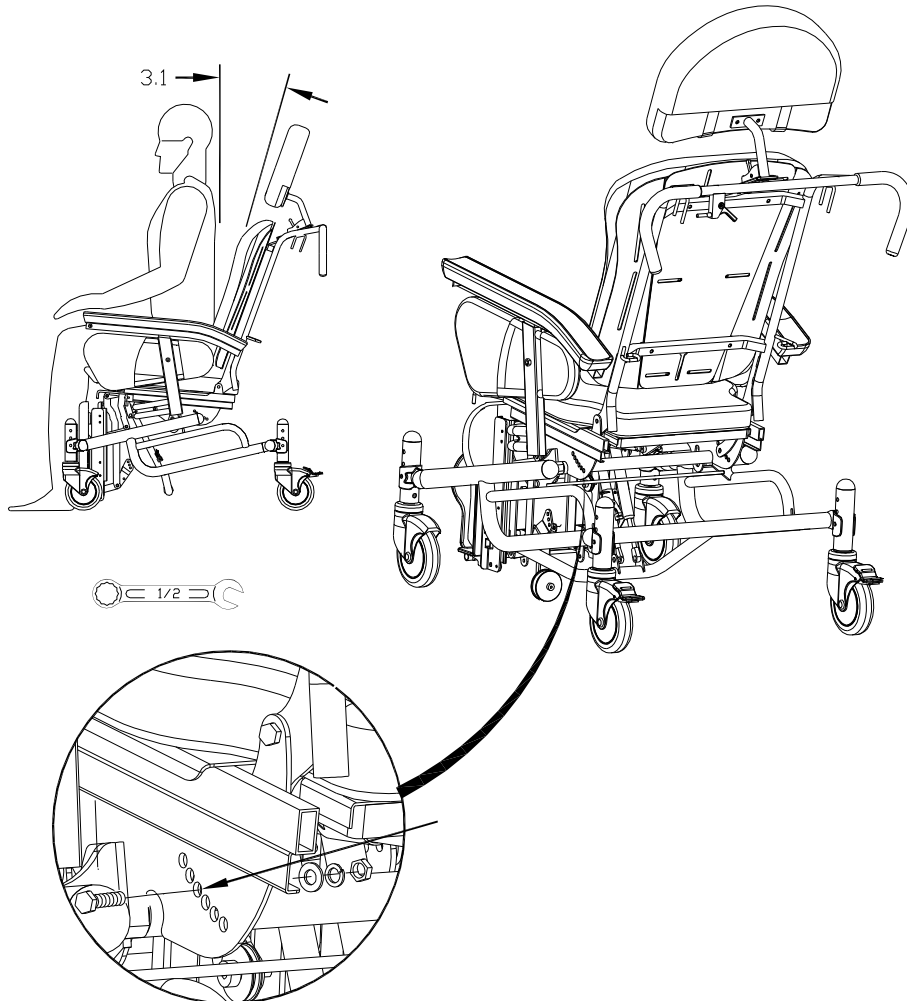
16. 19" wheel Seat height adjustment .

- 16.1. Determine if the seat height should be adjusted up or down.
- 16.2. When the chair is unoccupied, unscrew and remove the 19" wheel's (4 bolts).
- 16.3. Determine the desired height and attach the structure with the 19" wheels to the wheel support and tighten it with sets of bolts, nuts and washers.

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
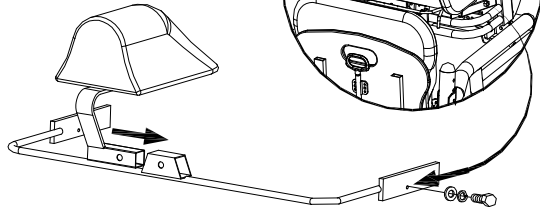
14. Foldaway perinial abductor pad.

- 14.1 Slide the foldaway perinial abductor pad support in the cavity.
- 14.2 Tighten the two screws.
- 14.2 Slide the foldaway perinial abductor pad in its support and press in the release pressure lock button.

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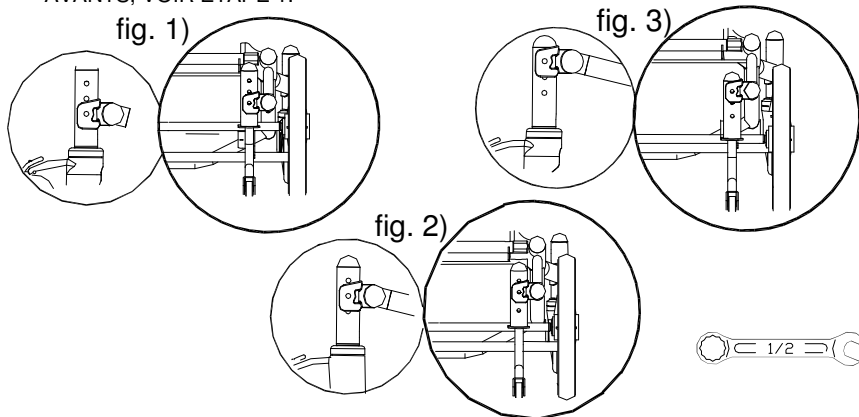15. 13" wheels seat adjustment.

- 15.1. Determine if the seat height should be adjusted up or down.
- 15.2. When the chair is unoccupied, unscrew and remove the 13" wheel's bolts attach. (2 bolts)
- 15.3. Determine the desired height and attach the structure with the 13" wheels to the cylindrical support and tighten it with sets of bolts, nuts and washers supplied.

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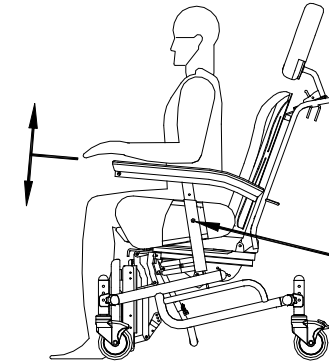
4. Armrest height.

- 4.1. Determine if the armrest height needs adjustment.
- 4.2. Remove the bolt with plastic cap on the armrest side and set the desired height. Replace the bolt.
- 4.3. BE SURE EACH SIDE IS AT THE SAME LEVEL. Check the height for proper fit and modify if necessary.

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

5. Armrest width.

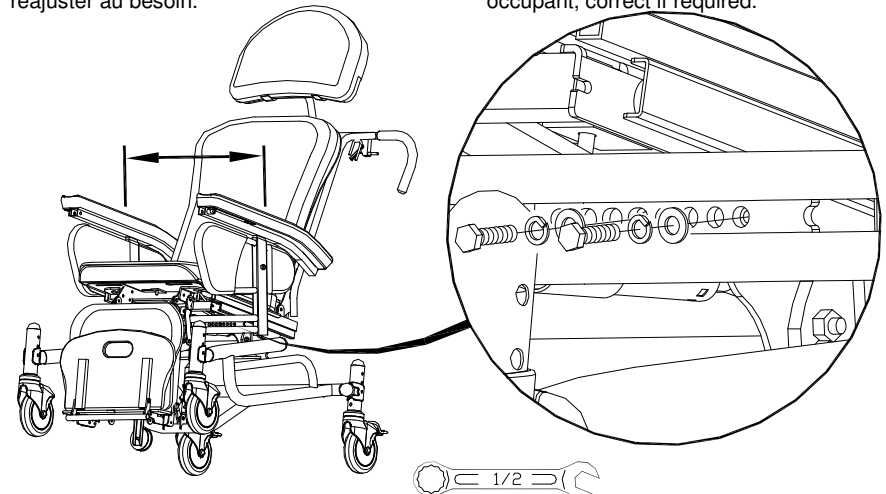- 5.1. When the chair is unoccupied, determine if the armrest width needs adjustment.
- 5.2. Remove the 2 bolts at the armrest side. Move the armrest left or right to the desired width. Replace the 2 bolts and the lock washers.
- 5.3. Check the seat is well adapted for the occupant, correct if required.

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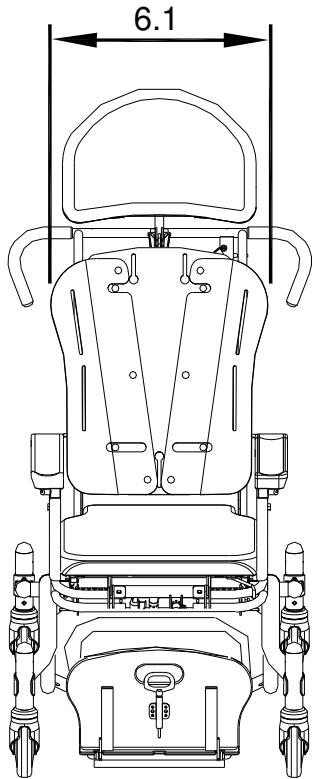
FRONT VIEW
VUE DE FACE

6. Backrest width.

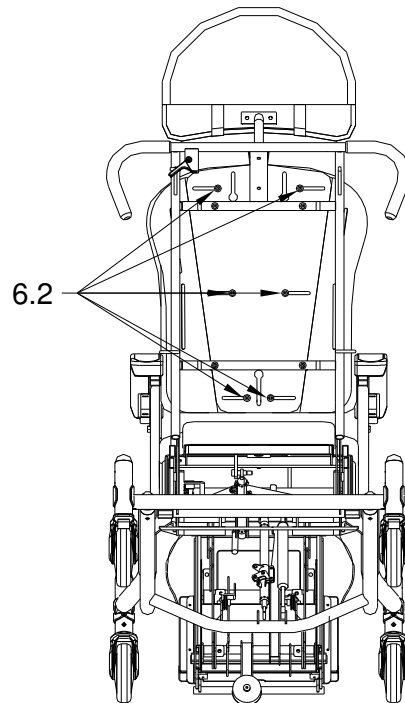
- 6.1. When the chair is unoccupied, determine if the backrest width needs adjustment.
- 6.2. Loosen the 6 screws on the back of the backrest and slide the plate to the desired width.
- 6.3. BE SURE EACH SIDE IS AT THE SAME WIDTH. Check the seat is well adapted for the occupant, correct if required.

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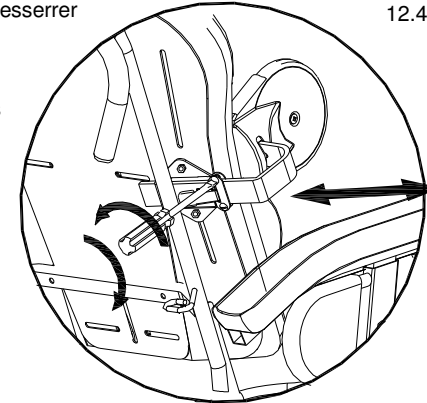
- 12.4. Check if the chair is well adapted for the occupant, correct if required.

12. Lateral and shoulders supports width adjustment.

- 12.1. Determine if the lateral or shoulders supports need adjustment.
- 12.2. Slightly loosen the 2 adjustment lock screws with a screwdriver. DO NOT REMOVE.
- 12.3. Move the lateral supports to the desired position BE SURE EACH SIDE IS AT THE SAME WIDTH and retighten the screws.

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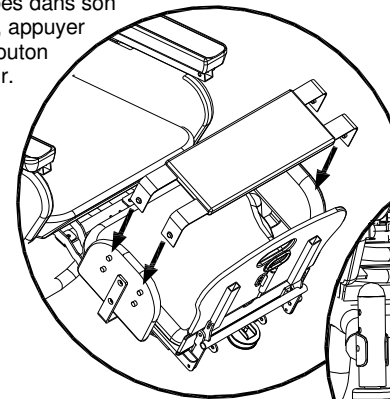
13. Lateral legs protectors.

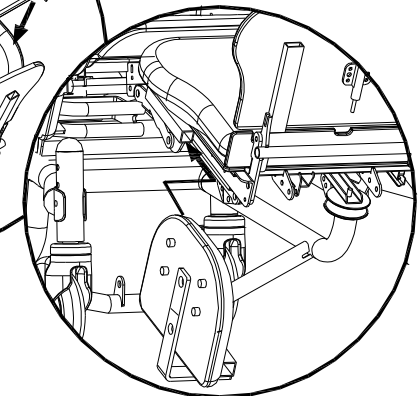
- 13.1. Press in the pressure release lock button, then slide the lateral legs protectors into its support.
- 13.2. Secure the legs protectors cushion with the pressure snaps (2 snaps on each side).

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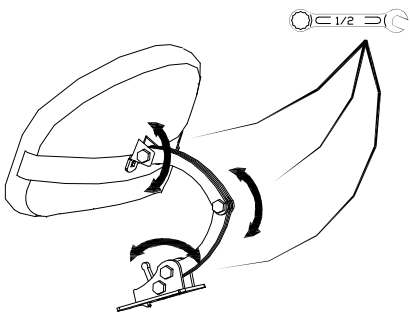
10. Adjustable Headrest.

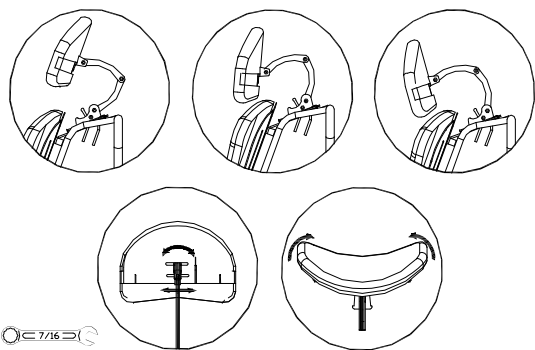- 10.1. Determine the desired position of the headrest.
- 10.2. Slightly loosen the adjustment bolts, DO NOT REMOVE, adjust the headrest and re-tighten the bolts..
- 10.3. The plate can be bent to fit with the patient head shape.
- 10.4. Check if the chair is well adapted for the occupant, correct if required.

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
11. Lateral and shoulders supports height adjustment.

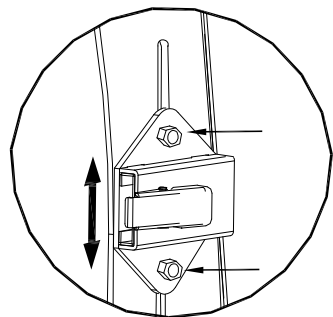- 11.1. Determine the correct position of the lateral or the shoulders supports.
- 11.2. Slightly loosen the adjustment bolts, DO NOT REMOVE.
- 11.3. Adjuste each side to the same level and re-tighten the adjustment bolts.
- 11.4. Check if the lateral and shoulders support is well adapted for the occupant, correct if required.

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

7. Seat width.

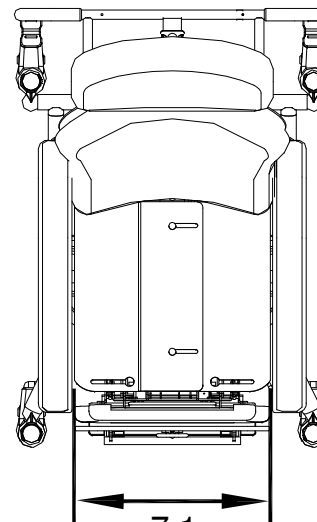- 7.1. When the chair is unoccupied, determine if the seat width needs adjustment.
- 7.2. Loosen the screws under the seat and slide the plates to the right width.
- 7.3. BE SURE EACH SIDE IS AT THE SAME WIDTH. Check if the chair is well adapted for the occupant, correct if required.

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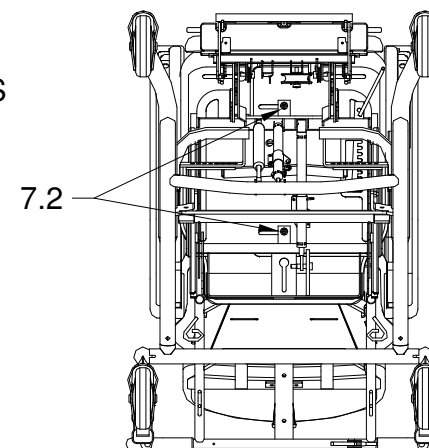8. Footrest height.

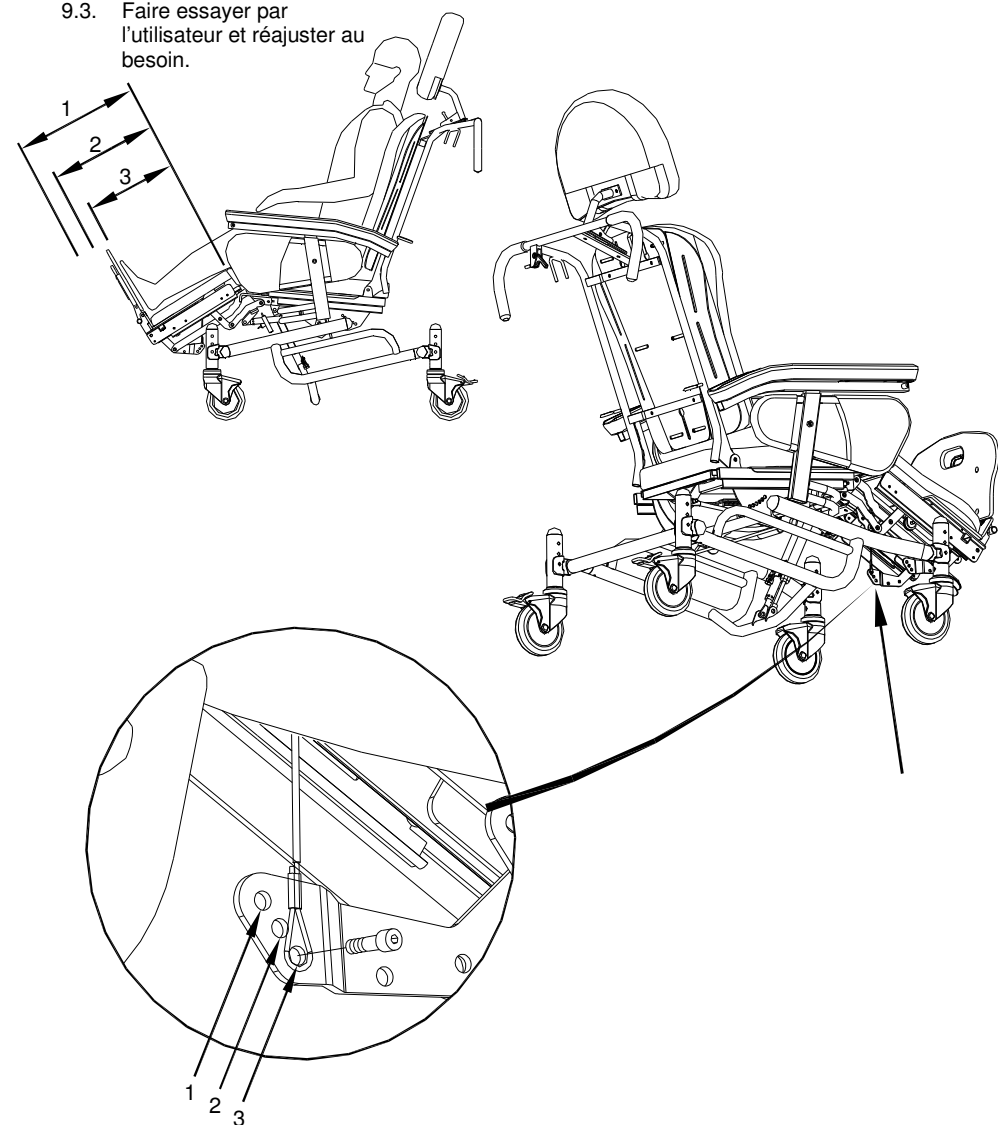
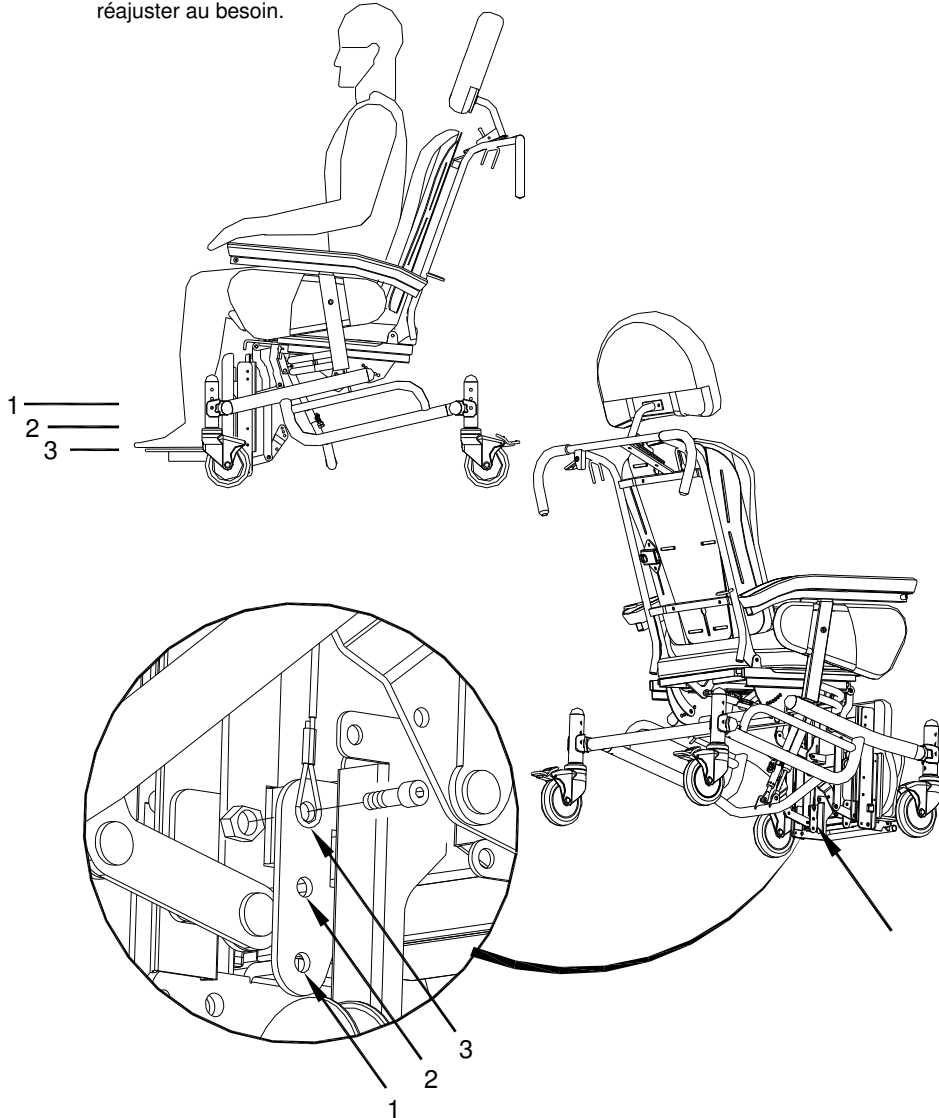
- 8.1. When the chair is unoccupied, determine if the footrest height needs adjustment.
- 8.2. On each side of the footrest remove the insert cables bolt. BE SURE EACH SIDE IS AT THE SAME LEVEL AND REPLACE THE BOLTS AND LOCK WASHERS.
- 8.3. Check if the chair is well adapted for the occupant, correct if required.

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9. Footrest extension.

- 9.1. When the chair is unoccupied, determine if the footrest length needs adjustment. Pull the handle below the footrest to correct the length.
- 9.2. Remove the bolt of each side of the footrest, extend it and replace the bolt. BE SURE EACH SIDE IS AT THE SAME LEVEL.
- 9.3. Check if the chair is well adapted for the occupant, correct if required.

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TERMS & CONDITIONS

GENERAL:

• All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

• LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery, or within 5 days if it is subsidized damage.

RETURN GOODS

• Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request must include the following information:
Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY: 3-1

• LPA MEDICAL warranty covers all defects in workmanship or materials for the following periods of time. From date of purchase:

Metal parts (frame) 5 years

Mechanism: 3 years

Controls, tray, cylinder, cushions, pads & pillows (fill material, fabric covers) 1 year

If service required is order this warranty, contact our Customer Service Department in writing, giving the following information

1- Model or part number involved

2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number

3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

• LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric facing and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, your may also have other rights witch vary from Province to Province or State to State.

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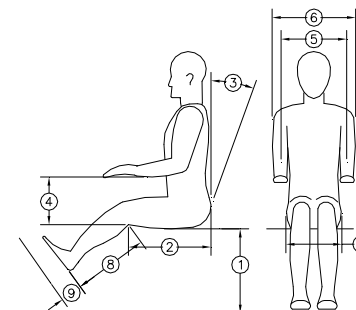
WARNING

All brakes MUST BE ENGAGED AT ALL TIME except during transportation.

This product is equipped with a mechanized legrest. DO NOT let children sit on this chair without supervision. DO NOT let children operate any mechanism. A child could be seriously injured when declining the legrest. ALWAYS leave an empty chair in the upright position.

Keep hands and feet clear of all mechanisms.

The chair should be operated by a healthcare professional.

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment and under constant supervision.

Comp-Position Maintenance

For optimum performance, it is necessary that the chair be kept clean and all parts adjusted properly.

When needed, the upholstered surfaces may be washed with a mild detergent diluted or dissolved in water. If necessary, a mild, clear disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as described above.

Possible adjustments of the chair

1. See step 1
2. See step 2
3. See step 3
4. See step 4
5. See step 5
6. See step 6
7. See step 7
8. See step 8
9. See step 9