HARMONY HARMONIE

Care-Recliner
Model C-207

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LPA MEDICAL INC.

460, Desrochers, #150, Vanier, Québec, Canada, G1M 1C2

Tel: (418) 681-1313, (800) 663-4863

Fax : (418) 681-4488

lpa@lpamedical.com <http://www.lpamedical.com>

1940, Country Club Road, Long Lake, MN 55356 USA

Tel: (888) 845-6447

Fax: (952) 745-0197

<http://www.optimaol.com>

HARMONY HARMONIE

Care-Recliner
Model C-207

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Assembly and Operation

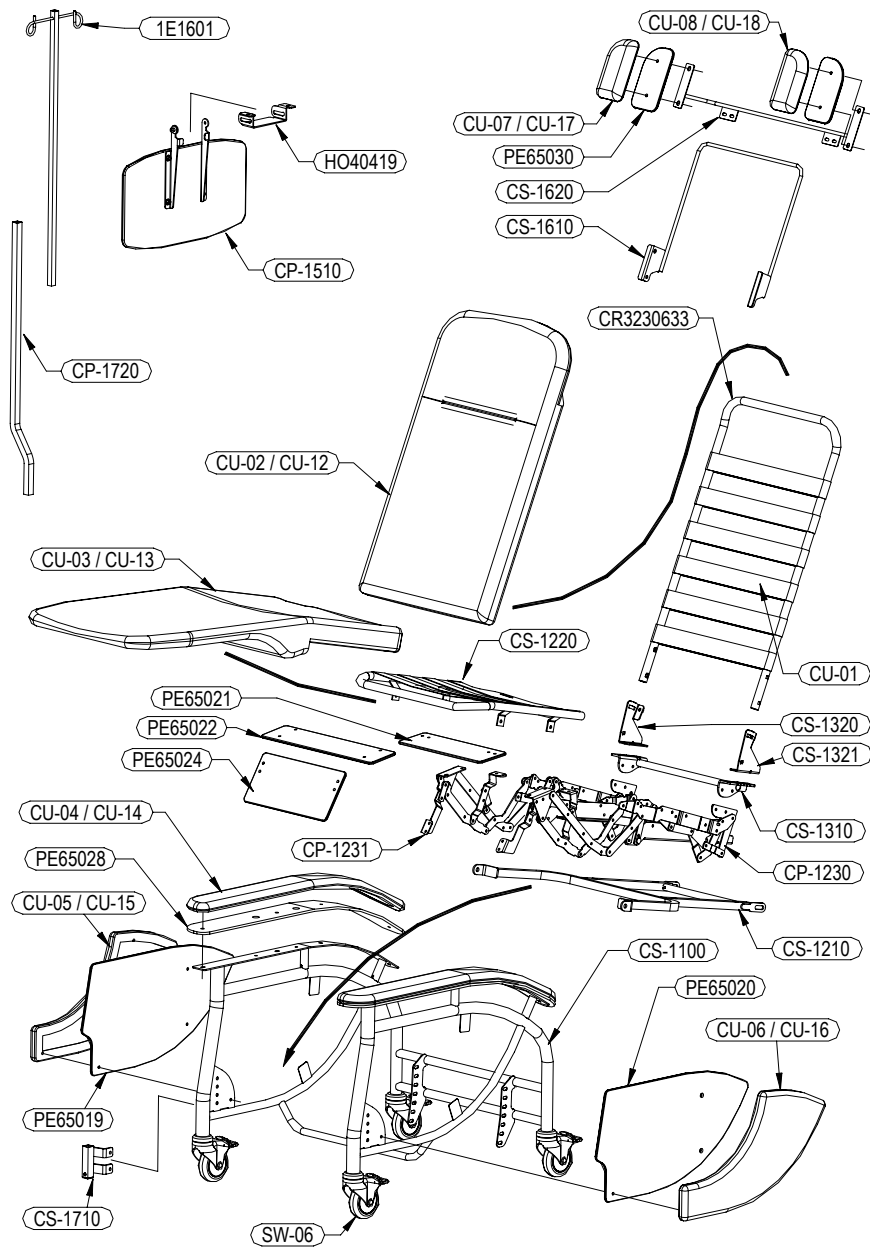
Your HARMONY care-recliner is built for years of trouble-free service. Carefully read and follow the instructions step-by-step to assemble, operate and care for your recliner.

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
Vue explosée

Chair Adjustments

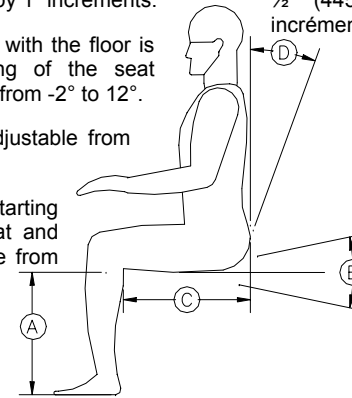- A. The seat height from floor is adjustable from 17 ½" to 20 ½", by 1" increments.
- B. The initial seat angle with the floor is adjustable. Depending of the seat height, it's adjustable from -2° to 12°.
- C. The seat depth is adjustable from 17" to 18 ½".
- D. The angle of the starting position between seat and backrest is adjustable from 5° to 18°.

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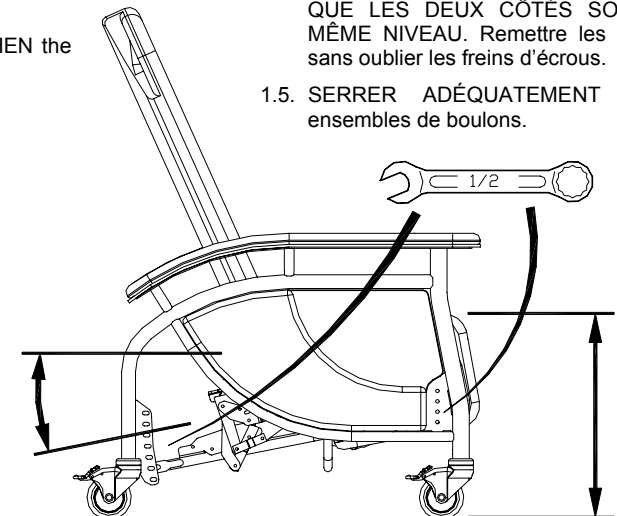
1. Seat height (A) and Seat Angle Adjustments (B).

- 1.1. Determine if the seat height should be adjusted up or down.
- 1.2. Loosen the 4 screws (2 at front and 2 at rear) of the seat structure. **DO NOT REMOVE.**
- 1.3. Remove the 2 front screws and move the seat structure to the desired position. **BE SURE EACH SIDE IS AT THE SAME LEVEL.** Replace the bolts with the lock washers.
- 1.4. Remove the 2 rear screws and move the seat structure to the desired position to obtain the correct seat angle. **BE SURE EACH SIDE IS AT THE SAME LEVEL.** Replace the bolts with the lock washers.
- 1.5. **FIRMILY THIGHTEN** the 4 screws.

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
2. Seat Depth Adjustment (C)

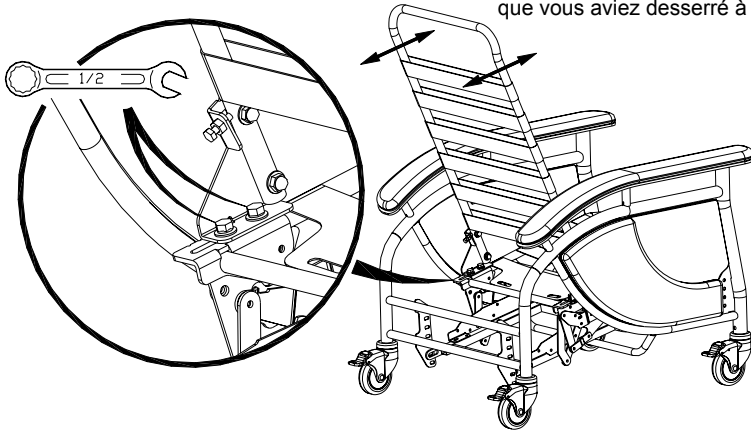- 2.1. Loosen (DO NOT REMOVE) the 2 sliding bolts on the bracket on each rear side of the seat as shown.
- 2.2. Adjust the seat depth correctly and tighten the bolts. Adjust each side to the same depth.
- 2.3. Tighten the bolts loosened in 2.1.

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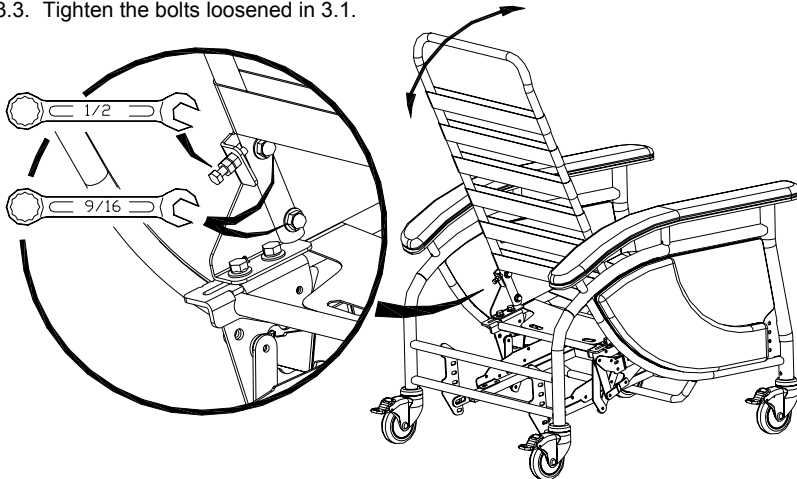
- 3.1. Loosen (DO NOT REMOVE) the 2 bolts on the bracket at each rear side of the backrest as shown below.
- 3.2. To adjust the backrest angle, loosen or tighten the bolts at the rear of the bracket until the desired angle is achieved.
- 3.3. Tighten the bolts loosened in 3.1.

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
Accessories

Lateral Tray (right or left)

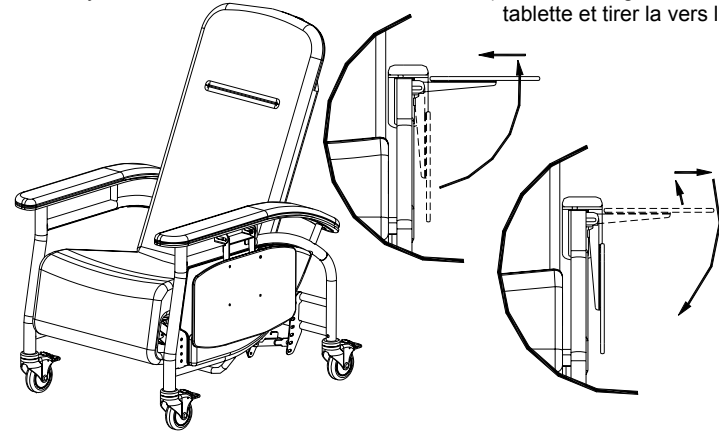- a) Fold up the tray to the horizontal position and push back to secure it.
- b) To fold down, raise it a slightly and pull out the tray.

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
Elbows Supports Adjustment

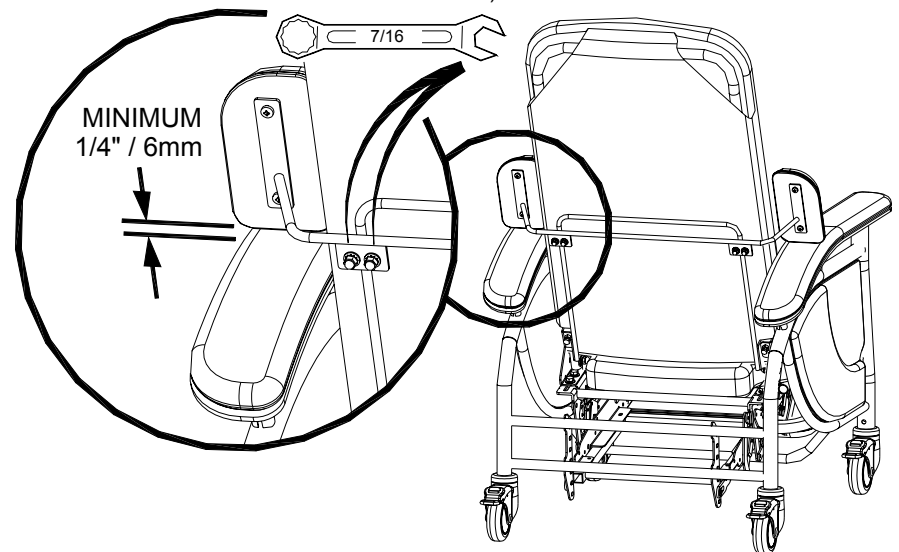- a) Loosen (NOT REMOVE) the 2 U-bolts on the elbows support.
- b) Adjust it to minimum 1/4" over the armrest on each side when the chair is closed.
- c) Tighten the U-bolts on each side.

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TERMS & CONDITIONS

GENERAL:

- All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

- LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsided damage.

RETURN GOODS

- Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request must include the following information:
Reason for requested return
LPA MEDICAL invoice or acknowledgement number
LPA MEDICAL model or part number and number of cartons and items being returned.
Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY: 3-1

- LPA MEDICAL warranty covers all defects in workmanship or materials for the following periods of time.
From date of purchase:
-Metal parts (frame) 3 years
-Mechanism: 3 years
-Upholstery, tray, wheel (fill material, fabric covers): 1 year
If service required is order this warranty, contact our Customer Service Department in giving the following information
1- Model or part number involved
2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

- LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric facing and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, your may also have other rights which vary from Province to Province or State to State.

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WARNING

All brakes **MUST BE ENGAGED AT ALL TIMES** except during transportation.

This product is equipped with a mechanized legrest. **DO NOT** let children sit on this chair without supervision. **NOT** let children operate any mechanism. **A child could be seriously injured when declining the legrest.**

ALWAYS leave an empty chair in the upright position.

Keep hands and feet clear of all mechanisms.

ONLY the occupant should operate the chair.

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment and under constant supervision.

HARMONY Maintenance

For optimum performance, it is necessary that the chair be kept clean and all parts adjusted properly.

When needed, the upholstered surfaces must be washed with a mild detergent diluted or dissolved in water. If necessary, a mild, clear disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as described above.

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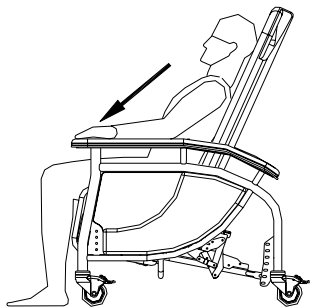
How to use of your HARMONY


All brakes **MUST BE ENGAGED AT ALL TIMES**, except during transportation.

1. To open the leg rest, sit in the chair and take support on the end of the armrest to push back your body. This will activate the mechanism and open the leg rest.


2. To incline the backrest, lie back into the chair.

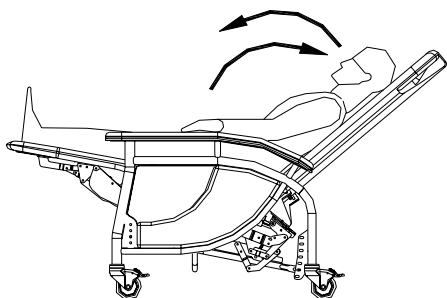3. To decline the backrest, move back into the upright position.

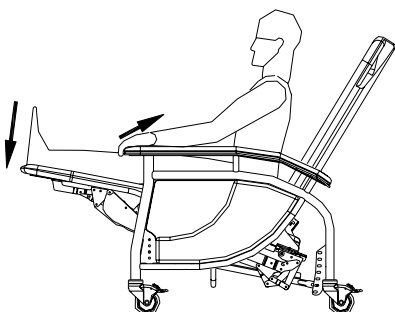
4. To decline the leg rest, push it down with your feet. At the same time, grip the armrest and pull forward.

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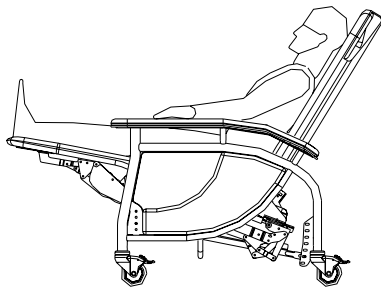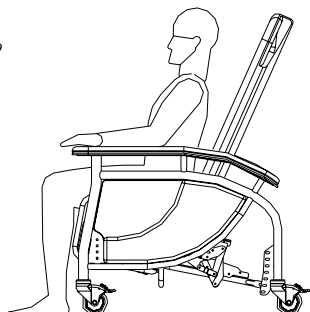
Accessory Part list	
CP-1510	Lateral tray
CS-1610	Elbows support Height frame adjustment
CS-1620	Elbows support cushions
CS-1710	I/V pole support
CP-1720	I/V pole lower part
1E1601	I/V pole higher part
HO40419	Lateral tray support
PE65030	Elbows support plastic protector
CU-07	Vinyl RIGHT Elbow cushion C
CU-08	Vinyl LEFT Elbow cushion C
CU-17	Crypton RIGHT Elbow cushion C
CU-18	Crypton LEFT Elbow cushion C

Important: Note « C »
Always specify fabric name, pattern, color, and the HARMONY's serial number for all replacement part orders. Upholstered replacement parts may vary slightly in color from the original part due to dye lot differences. LPA Medical will not be responsible for dye lot differences.

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Using the HARMONY Chair

For the safest operation, please familiarize yourself with the following components:

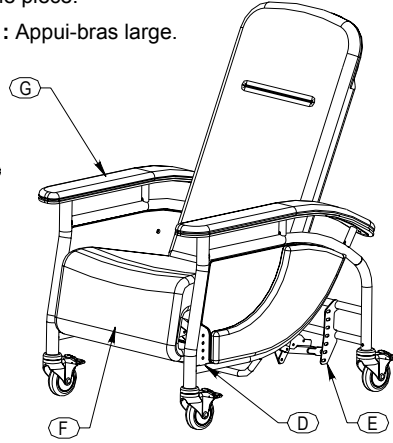
- A:** Swivel wheels with brake.
- B:** Seat depth and backrest angle adjustment.
- C:** Push bar.
- D:** Front seat height adjustment.
- E:** Rear seat height adjustment.
- F:** One piece seat and leg rest cushion.
- G:** Wide armrest.

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Available Accessories

- 1:** Fold-down side tray (left or right).
- 2:** Adjustable elbow supports.
- 3:** Cervical support.
- 4:** I/V pole support.
- 5:** Adjustable I/V pole.

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FOR YOUR RECORDS

Purchaser's record for LPA MEDICAL 3-1 WARRANTY

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THANK YOU FOR CHOOSING LPA MEDICAL PRODUCTS.

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PLEASE FILL OUT THIS INFORMATION CARD, DETACH AND RETURN IT WITHIN 30 DAYS OF PURCHASE.

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Facility's Street Address: _____ : Adresse du centre _____

City: _____ : Ville _____

State / Province: _____ Zip: _____ : Code postal _____ : Province _____

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What influenced your purchasing decision?

- Magazine Ad
- Direct mail
- Brand name
- Convention exhibit
- Salesperson's suggestion
- Other _____

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